

THANKS FOR PURCHASING THE DYNAMIC BAG FROM HANDY BAG!

Installation Process

1) Unclip the straps on the carrier and loosen the strap to make it long enough to go over each side of your back rest posts.

2) The magnet mounts on each of the carrier straps are adjustable. You will need to adjust their position once you have mounted the carrier to the back rest posts. Simply slide the magnet mounts so they line up with the connecting magnet mounts on the bag itself.



3) Once you have your male carrier magnet mounts in position, clip the carrier straps into their buckles at the bottom of the carrier and then pull down and toward the center of your back rest to tighten equally so the carrier is centered on your back rest.

4) Once you have secured the carrier bag to your wheelchair, slide the Dynamic Bag inside the carrier and double check the placement of the magnet mounts.

Your Dynamic Bag should be in place and ready to tackle the world! Please be sure to review the Dynamic Bag on our website and let us know if you have any questions.



 **Living Spinal**

email: info@livingspinal.com

phone: 619-810-0010